

Description

This kit includes a backrest adjusting lever for use with the F2 Monobloc Ambulance Cot.

This manual includes parts and instructions for removing and installing a backrest adjusting lever.

Tools Required	Qty.
5 mm Hex Wrench	1 ea.
10 mm Open End Wrench	1 ea.


Parts Supplied	Qty.
Backrest Adjusting Lever	1 ea.
Spring	1 ea.
Screw	1 ea.
Washer	2 ea.
Lock Nut	1 ea.

WARNING

Improper installation and use of the kit can cause injury. Install and use the kit only as described in this manual.

Improper parts and service can cause injury. Use only Ferno parts and Ferno-approved service on the product.

Modifying the kit can cause injury and damage. Use the kit only as designed by Ferno.

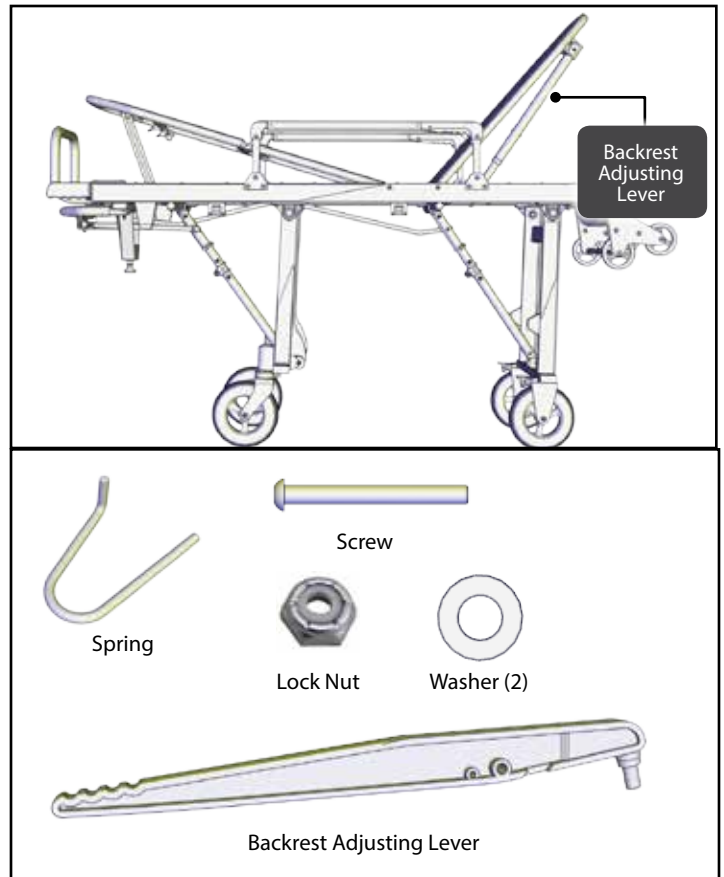


Removing the Backrest Adjusting Lever

1. Raise the backrest to its highest position.
2. Using the 5 mm hex wrench and the 10 mm open end wrench, remove and discard the backrest adjusting lever, spring, screw, washers and lock nut (Figure 1).

Tip

Removing the Backrest Adjusting Lever

Compress the lever against the backrest tube to relieve tension. This helps with removing the screw.

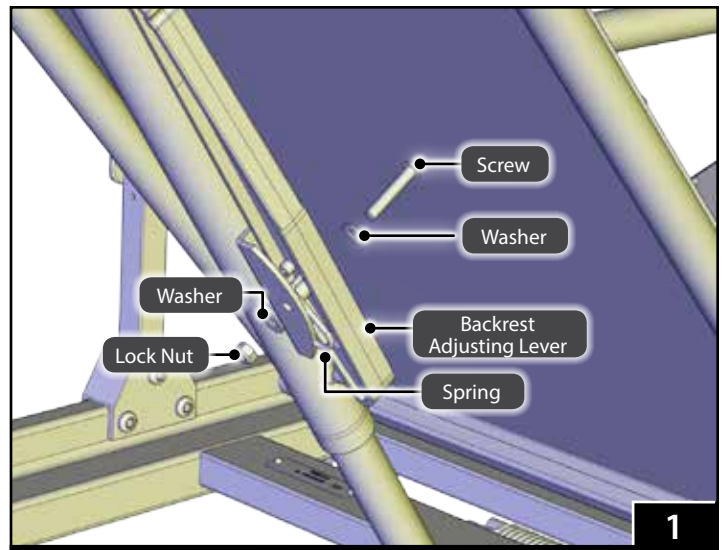



Installing the New Backrest Lever

1. Insert the 90 degree leg of the spring into the backrest adjusting lever (Figure 2). The long end of the spring is held by the slot.
2. Hold the spring in place. Compress the long end of the spring against the backrest tube and align the screw holes.
3. Place the lever assembly into the mount (Figure 3). Hold in place as you complete step 4.

Tip


Installing the Backrest Adjusting Lever

Start high in the backrest tube bracket, then slide down.
Use an assistant as necessary.

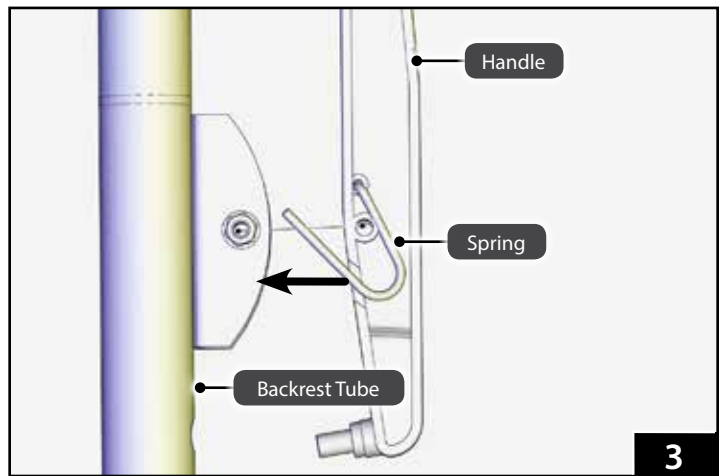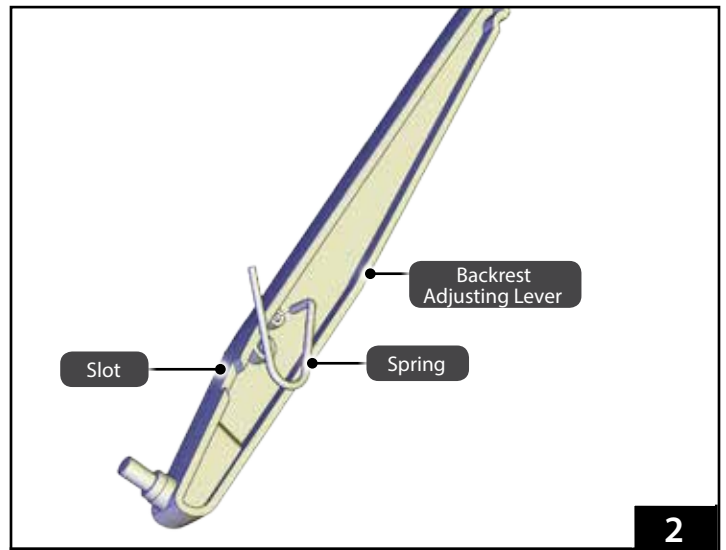

F2 Backrest Adjusting Lever Kit

- Place a washer on the screw and insert the screw through the lever assembly as shown in Figure 4.
- Place the second washer on the other end of the screw and thread the lock nut (Figure 4).
- Tighten the screw and lock nut.
- Check the function of the new backrest adjustment lever. Confirm the backrest can be adjusted to each locking position.

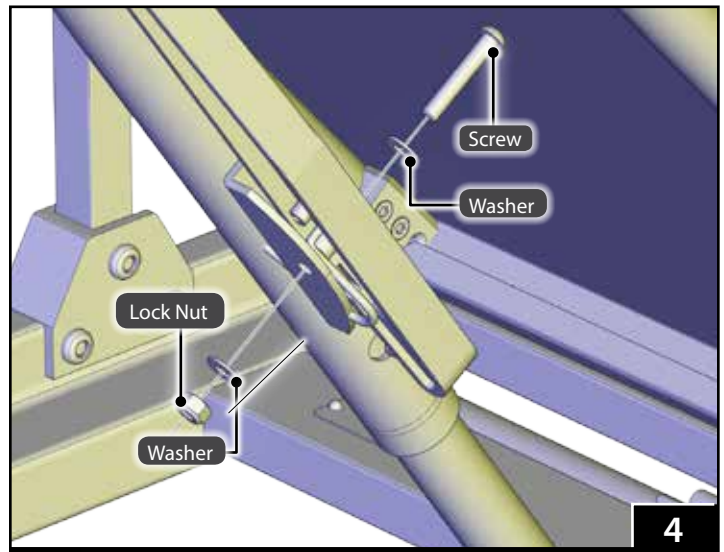

Repairs or Parts

EMSAR® is the only organization authorized by Ferno to manage, service, and repair Ferno products.



Telephone (Toll-free)	1.800.73.EMSAR
Telephone (Worldwide)	1.937.383.1052
Fax (Worldwide)	1.937.383.1051
Internet	www.EMSAR.com

Ferno Customer Relations

For ordering assistance or general information:

CANADA AND THE U.S.A.

Telephone (Toll-free)	1.877.733.0911
Telephone	1.937.382.1451
Fax (Toll-free)	1.888.388.1349
Fax	1.937.382.1191
Internet	www.ferno.com

ALL OTHER LOCATIONS

For assistance or information, please contact your Ferno distributor. If you do not have a Ferno distributor, please contact Ferno Customer Relations:

Ferno-Washington, Inc., 70 Weil Way
Wilmington, Ohio 45177-9371, U.S.A.


Telephone	+1.937.382.1451
Fax	+1.937.382.6569
Internet	www.ferno.com

Limited Warranty Statement

The products sold by Ferno are covered by a limited warranty, which is printed on all Ferno invoices. The complete terms and conditions of the limited warranty, and the limitations of liability and disclaimers, are also available upon request by calling Ferno at 1.800.733.3766 or 1.937.382.1451.



Ferno-Washington, Inc.
70 Weil Way
Wilmington, Ohio 45177-9371 U.S.A.
1.937.382.1451

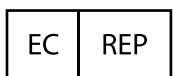

Ferno (UK) Limited
Stubs Beck Lane, Cleckheaton
West Yorkshire BD19 4TZ, United Kingdom
44 (0) 1274 851999